

Curera[®] Multi Pillow

Prevent pressure ulcers with knee & heel support



 **curera**[™]
positionering

 **care of sweden**

SUPPORTING LIFE

Curera® Multi Pillow

If a person makes too few spontaneous movements, there is an increased risk of pressure ulcers. Regular changes of position reduce the risk of pressure ulcers. Position as required in both reclining and seated position.

Curera® Positioning is a range of stable pillows, rolls, and wedges that meet most needs for support and relief. They are also a great help when changing dressings and for patient care. Our hygiene covers and cotton covers are also available as accessories to the pillows and rolls.

The hygiene cover does not allow liquid or air to penetrate, but is vapour permeable (vapour is wicked away). Make sure that the patient is positioned correctly so that airways are unobstructed.

This versatile pillow can be placed between the knees or under legs and heels for support and relief.

Working with Curera® Positioning is simple. Contact Care of Sweden for training and further information.

POSITIONING EXAMPLE



As support in lateral position.



Counters heel pressure against surfaces, supports the feet.



As relief for the heels.



In locked position in connection with spasticity.



As support under the knees.

CLEANING

The hygiene cover can be wiped clean with detergent and/or disinfectant. Machine wash at max. 95 °C.

Inner pillow: Machine wash at max. 95 °C. Can be tumble-dried or dried in an airing cupboard; avoid over-drying. For faster drying, run the spin cycle three or four times.

Cotton cover: Machine wash at max. 70 °C.

FACTS

- **Filling:** 50 % HR foam/ 50 % polyester fibre
- **Warranty:** 1 year
- **Sizes:** 45x45, 60x70 cm

Accessories:

Mattress covers: Polyurethane/polyester
Cotton cover



Always read the instruction for use prior to usage of the product.

Care of Sweden and Curera are registered trademarks of Care of Sweden AB. © Care of Sweden AB, 2020. All rights reserved. Reproduction prohibited. Last updated: 2021-05-31